

BRUGERMANUAL

CANTILEVER GRAB RAIL

Prod. no. 3840/3841/3842/3843/3844/3845





TABLE OF CONTENTS

Dimensions and technical specifications 3

General safety information 4

Installation 5

User guide 6

Cleaning and maintenance 6


Troubleshooting and red corrective actions 6

IMPORTANT

- This product must not be treated as household waste
- The product must be recycled where possible
- Seek advice from local authorities
- The product can also be returned to the manufacturer / distributor, who will ensure proper recycling / disposal

DIMENSIONS

Dimensions/unit of measurement: mm

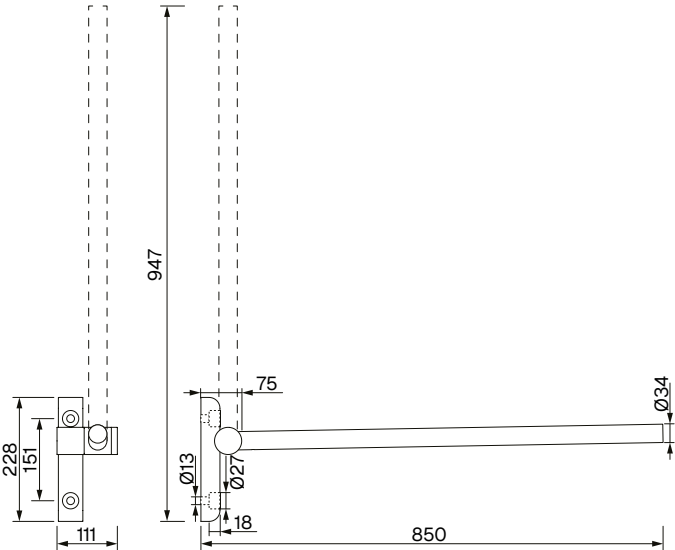

TECHNICAL SPCIFICATIONS

Description	Data
Dimensions, folded up	H: 947 mm x B: 111 mm x D: 75 mm
Dimensions, folded up	H: 228 mm x B: 111 mm x D: 850 mm
Material	Brushed stainless steel, satin finish
Load capacity	100 kg
Net weight	6,35 kg
Gross weight	8,15 kg
Model	3840 Cantilever grab rail, left-oriented, stainless steel
	3842 Cantilever grab rail, left-oriented, RAL classic
	3844 Cantilever grab rail, left-oriented, PVD coated
	3841 Cantilever grab rail, right-oriented, stainless steel
	3843 Cantilever grab rail, right-oriented, classic
	3845 Cantilever grab rail, right-oriented, PVD coated

GENERAL SAFETY INFORMATION


WARNING

- Always check that the grab bar is correctly and securely mounted before use
- Do not use the grab bar if it is loose, damaged, or shows signs of wear
- Do not exceed the maximum load of 100 kg
- The grab bar must only be used for its intended purpose
- Children must not play with the product


INSTALLATION

The cantilever grab rail must be wall-mounted. When selecting screws for wall fixing, the wall type must be taken into account. Consult your local hardware store for advice on choosing the appropriate fixings for the specific wall type


1. Mark two holes vertically, with a distance of 151 mm, in the desired position on the wall.

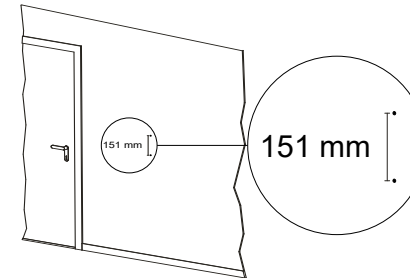


2. Drill holes for installation.

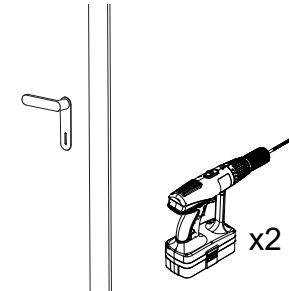


3. Insert wall plugs/anchors into the wall.

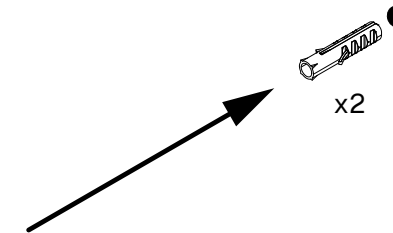

4. Place the cantilever grab rail on the wall and secure it with screws suitable for the wall type.

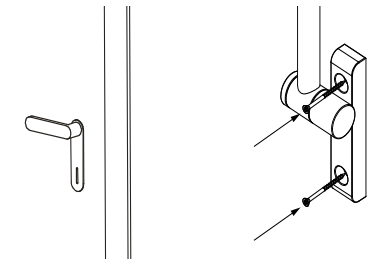

5. Place the cover caps over the mounting holes.

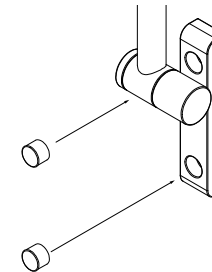

6. The cantilever grab rail is ready for use.





USER GUIDE

- Grasp the cantilever grab rail with your whole hand
- Apply steady, even pressure without jerking
- Use the grab bar to assist with balance and movement

CLEANING AND MAINTENANCE

- Do not rinse with water
- Clean with a damp cloth and a mild soap solution. Never use abrasive cleaners or cleaning products with a scouring effect

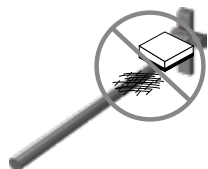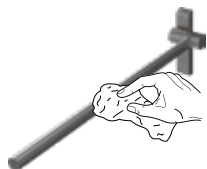

TROUBLESHOOTING AND CORRECTIVE ACTIONS

Contact DAN DRYER for assistance.



DAN DRYER A/S
Tåsingevej 2
DK - 8940 Randers SV

+45 8641 5711
info@dandryer.com
dandryer.com

