



Recommended Install Locations

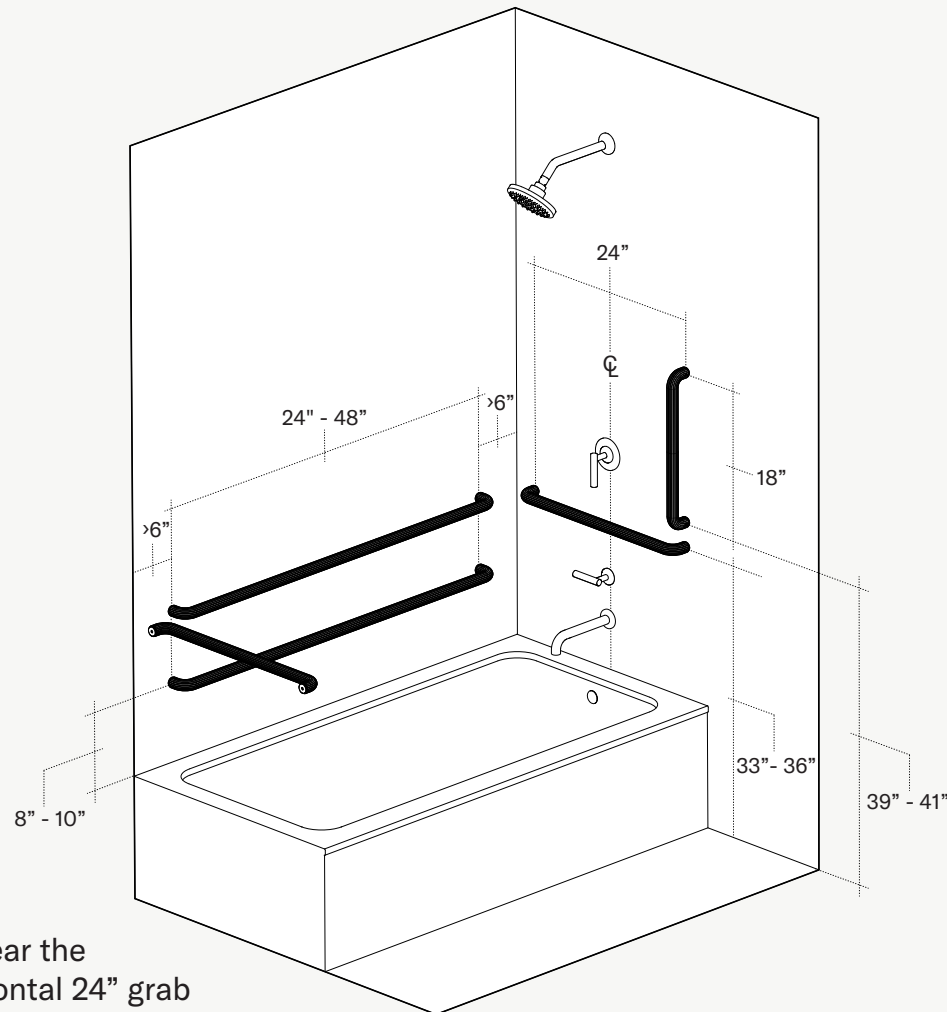

One (1) vertical 18" grab bar near the shower controls. Two (2) horizontal 24" grab bars situated on the short walls. One (1) horizontal grab bar between 24" & 48" situated on the long wall. Where there is a bath tub, install one (1) additional horizontal grab bar on the long wall.

Seek professional assistance for ADA compliant installation.


Recommended Install Locations

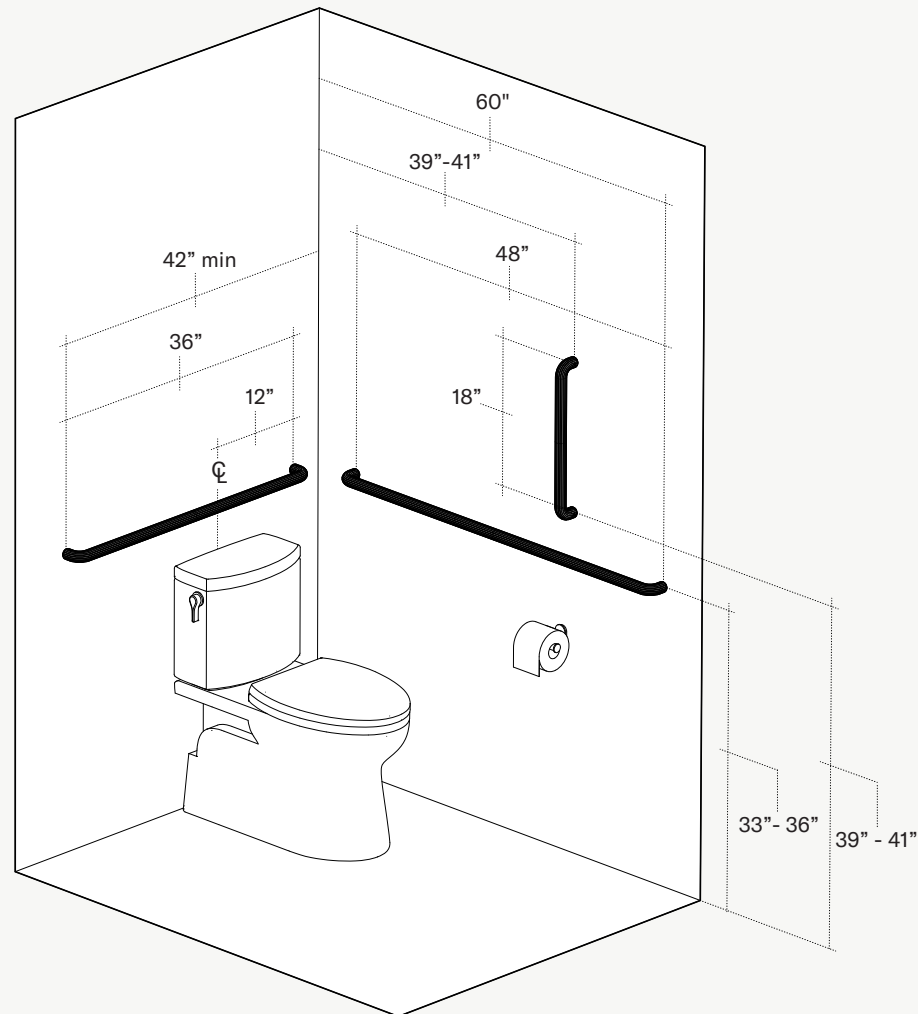

One (1) vertically oriented 18" bar and one (1) horizontally oriented 48" bar on the wall parallel to toilet. One (1) horizontally oriented 36" grab bar situated on the wall behind the toilet.

Seek professional assistance for ADA compliant installation.


Step #1

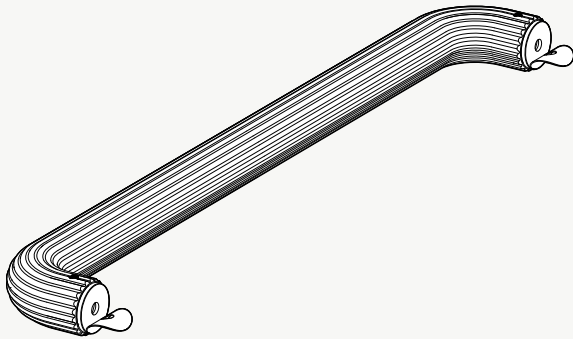

(Note: We recommend you seek the assistance of a professional to ensure proper & secure installation of your Remsen Grab Bars.)


Before install, ensure there is proper blocking and/or framing behind drywall at desired mounting location. Then remove film on mounting plate to reveal assistive adhesive.

Step #2

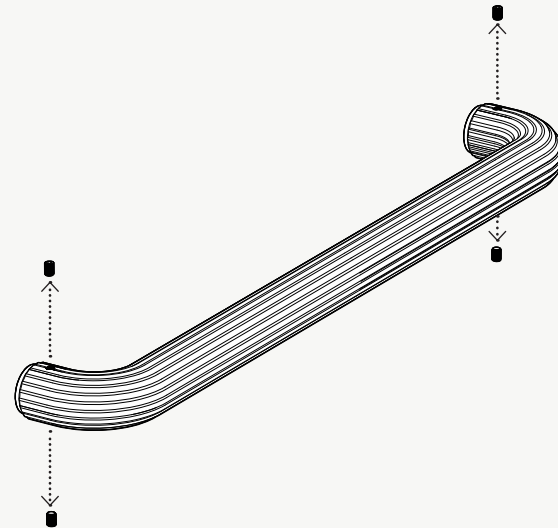


Remove set screws, and with mounting hardware still situated within the grab bar sockets, place mounting hardware backplates on the surface of desired locations.

Step #3

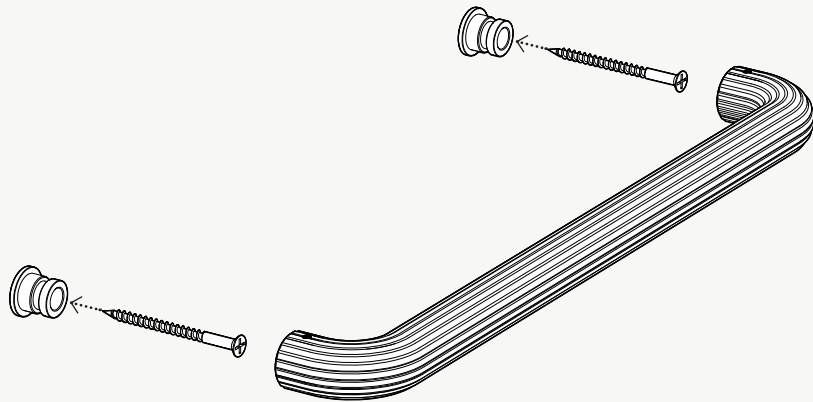


With mounting hardware in place, drive one #14 wood screw through each mounting hardware unit. Go through the wall and into the blocking or framing behind drywall.

Step #4

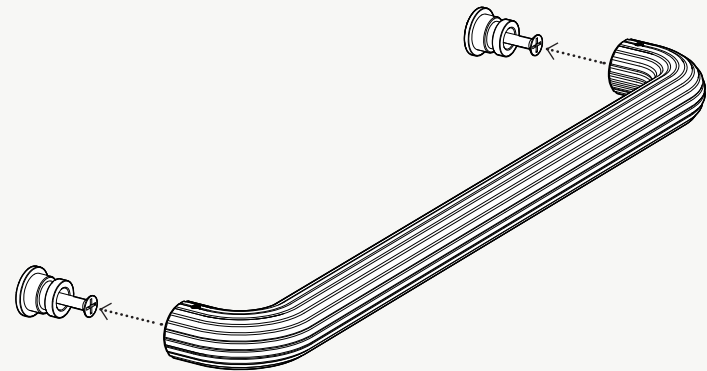


Drive screws until the mounting hardware is fastened tightly. Where there is no blocking or framing, use the appropriate drywall anchor. We suggest the Winglts Master Wall Anchor™ or the GeeFix Wall Anchor™.

Step #5

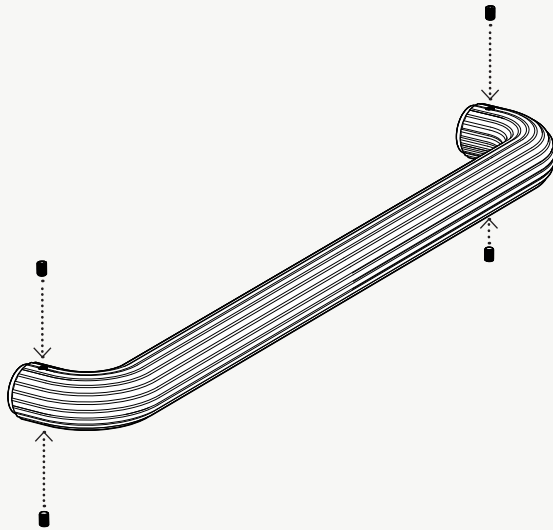

Place each of the grab bar sockets back onto their respective mounting hardware units. Then locate the supplied set screws and fasten each end with the help of the provided Allen wrench.

Step #6

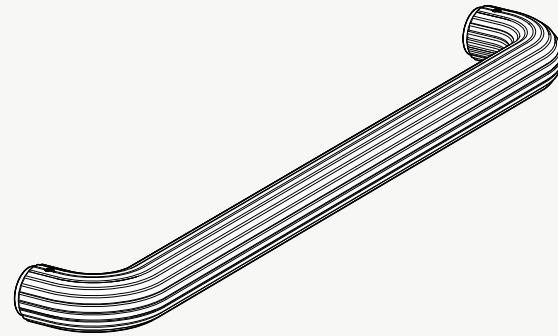

Consider it installed!